
Heraeus

PALAMIX®

Vacuum pump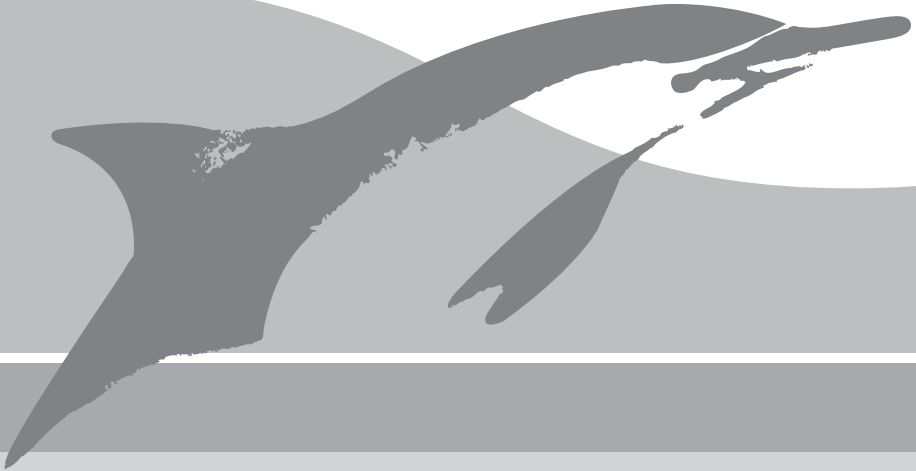

PALAMIX® Vacuum pump

EN	Instructions for use (Revision status: 2015-06)	4
EN (US)	Note: Instructions for use for sale within the US. Instructions for use (Revision status: 2018-04)	5
BG	Инструкции за употреба (Дата на актуализиране на текста: 2015-06)	6
CS	Návod k použití (Poslední revise: 2015-06)	7
DA	Brugsanvisning (Aktualiseret: 2015-06)	8
DE	Gebrauchsanweisung (Revisionsstand: 2015-06)	9
EL	Οδηγίες χρήσης (Ημερομηνία αναθεώρησης: 2015-06)	10
ES	Instrucciones de uso (Última revisión: 2015-06)	11
ET	Kasutusjuhend (Viimati muudetud: 2015-06)	12
FI	Käyttöohje (Revisiostatus: 2015-06)	13
FR	Notice d'utilisation (Date de révision: 2015-06)	14
HR	Upute za uporabu (Stanje revizije: 2015-06)	15
HU	Használati utasítás (Aktualizálás időpontja: 2015-06)	16
IT	Istruzioni per l'uso (Ultima revisione: 2015-06)	17
KK	Пайдалану нұсқаулары (Редакция күйі: 2015-06)	18
KO	사용 설명서 (2015년 06월 수정)	19
LT	Naudojimo instrukcija (Tikrinta: 2015-06)	20
LV	Lietošanas instrukcija (Pārbaudes statuss: 2015-06)	21
NL	Gebbruiksaanwijzing (Revisiestatus: 2015-06)	22

NO	Brugsanvisning (Revisjonsstatus: 2015-06)	23
PL	Instrukcja użytkowania (Ostatnia aktualizacja: 2015-06)	24
PT	Instruções de utilização (Data da última actualização: 2015-06)	25
RO	Instrucțiuni de utilizare (Stadiu revizie: 2015-06)	26
RU	Инструкция по применению (Ревизионный статус: 2015-06)	27
SK	Návod na použitie (Posledná revízia textu: 2015-06)	28
SL	Navodilo za uporabo (Datum revizije: 2015-06)	29
SV	Bruksanvisning (Revision status: 2015-06)	30
TH	คำแนะนำวิธีใช้ (แก้ไขปรับปรุงครั้งสุดท้ายเมื่อ: 2015-06)	31
TR	Kullanma Talimatı (Bilgilerin son güncellenme tarihi: 2015-06)	32
ZH	使用说明 (修订状态: 2015 年 06 月)	第33页

PALAMIX® vacuum pump

Description

The **PALAMIX® vacuum pump** is a compressed air operated vacuum pump. This generates a vacuum in the mixing cartridge of the **PALAMIX® vacuum mixing system** or the **PALACOS® R+G pro**, which allows mixing of bone cement under vacuum.

⚠ The **PALAMIX® vacuum pump** is located in the non-sterile area on the floor of the operating room.

Installation and connection

The **PALAMIX® vacuum pump** is connected with the in-house compressed air supply. The functional range of the pump is an operating pressure between 5 and 10 bar (0.5–1.0 MPa) air pressure. In order to allow the use of the **PALAMIX® vacuum pump**, it has to be connected to the mixing cartridge of the **PALAMIX® vacuum mixing system** or **PALACOS® R+G pro**. For this purpose, the respective vacuum hose supplied has to be connected via the activated charcoal filter to the **PALAMIX® vacuum pump**. The **PALAMIX® vacuum pump** can only be used in conjunction with the **PALAMIX® vacuum mixing system** or the **PALACOS® R+G pro**. The respective operating steps are also described here in detail.

Maintenance and cleaning

Regular maintenance of the **PALAMIX® vacuum pump** is not required. The pump housing can be cleaned with a soft cloth without aggressive detergents. Take care here that no cleaning fluid or water enters the **PALAMIX® vacuum pump**.

Notes

The **PALAMIX® vacuum pump** should not be used without a filter element. In the event of malfunctioning of the **PALAMIX® vacuum pump** contact your local Heraeus Medical representative.

PALAMIX® Vakuumpumpe

Note: Instructions for use for sale within the US.

Description

The **PALAMIX® vacuum pump** is a compressed air operated vacuum pump. This generates a vacuum in the mixing cartridge of the **PALAMIX® vacuum mixing system** or the **PALACOS® R+G pro**, which allows mixing of bone cement under vacuum.

Warnings

⚠ The **PALAMIX® vacuum pump** is located in the non-sterile area on the floor of the operating room.

Installation and connection

The **PALAMIX® vacuum pump** is connected with the in-house compressed air supply. The functional range of the pump is an operating pressure between 5 and 10 bar (0.5–1.0 MPa) air pressure. In order to allow the use of the **PALAMIX® vacuum pump**, it has to be connected to the mixing cartridge of the **PALAMIX® vacuum mixing system** or **PALACOS® R+G pro**. For this purpose, the respective vacuum hose supplied has to be connected via the activated charcoal filter to the **PALAMIX® vacuum pump**. The **PALAMIX® vacuum pump** can only be used in conjunction with the **PALAMIX® vacuum mixing system** or the **PALACOS® R+G pro**. The respective operating steps are also described here in detail.

Maintenance and cleaning

Regular maintenance of the **PALAMIX® vacuum pump** is not required. The pump housing can be cleaned with a soft cloth without aggressive detergents. Take care here that no cleaning fluid or water enters the **PALAMIX® vacuum pump**.

Precautions

The **PALAMIX® vacuum pump** should not be used without a filter element. In the event of malfunctioning of the **PALAMIX® vacuum pump** contact your local Heraeus Medical representative.

Caution

Federal law restricts this device to sale, distribution, and use by or on the order of a physician.

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PALAMIX® is a trademark of Heraeus Medical GmbH. Made in Germany.
Heraeus Medical GmbH