

palajet® HANDLING GUIDELINE


Preparation and use




1. Remove all components from the packaging and check that they are all present.




2. Suspend the battery pack and separate the suction hose from the saline connection as needed.

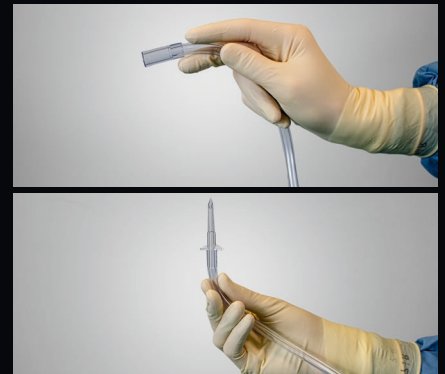

3. Connect the suction hose to the external suction device. Remove the protective cap from the saline connection and connect it to the irrigation solution.

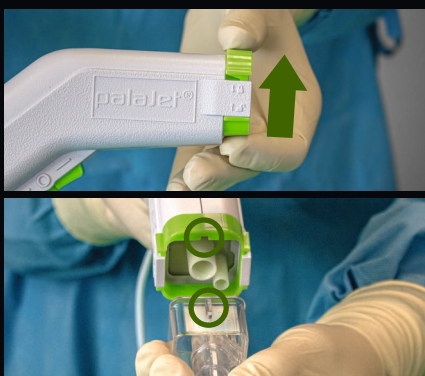


4. Slide the green lock on the handpiece up and insert the selected nozzle with the rib facing upward. Then push the lock down again.

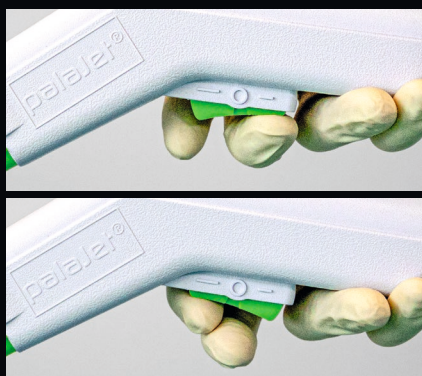

5. Activate the switch on the handpiece to start the irrigation process. Select between the following settings:

- Short bar = low irrigation pressure
- Circle = stop irrigation
- Long bar = high irrigation pressure

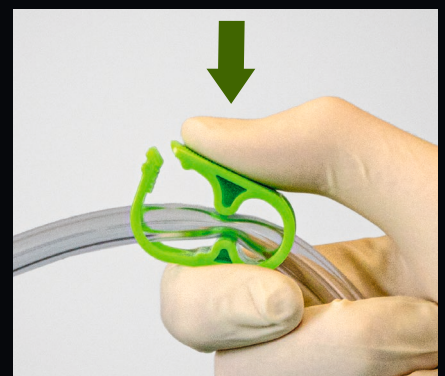

6. Optional: The clamp on the saline connection hose and suction hose can also be used to regulate the flow rate or suction.

- Green clamp = suction hose
- Blue clamp = saline connection hose


palajet®

HANDLING GUIDELINE


After use




1. Complete the application by placing the switch on the handpiece in the stop position 'O' and switching the external suction device off.

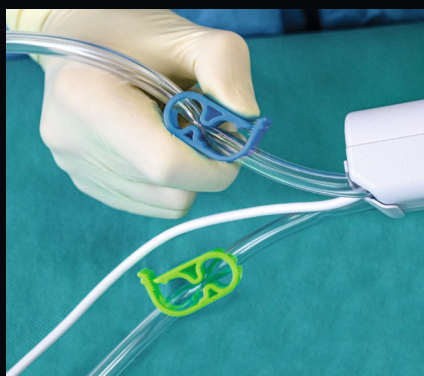

2. Close the clamp on the saline connection hose and suction hose to prevent return flow of liquids.

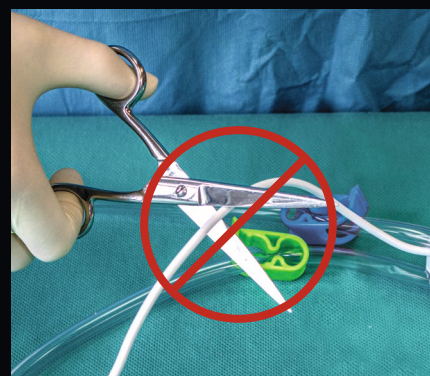

3. **CAUTION: Danger of short circuit!** Do NOT cut the cable to the battery pack.

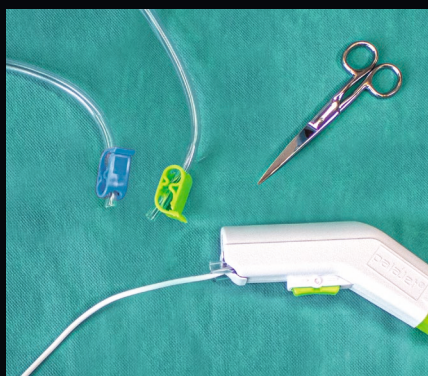

4. Cut the hoses off.



5. Open the battery pack by pressing down on at least one of the two tabs on the side and push off the cover.



6. Take out the battery bundle. Remove the batteries and dispose of them correctly.

