Heraeus

Medical

PALAMIX®

Vacuum pump

Instructions for use

PALAMIX® Vacuum pump

EN	Instructions for use (Revision status: 2023-09-18)	4
BG	Инструкции за употреба (Дата на актуализиране на текста: 2023-09-18)	6
CS	Návod k použití (Poslední revise: 2023-09-18)	8
DA	Brugsanvisning (Aktualiseret: 2023-09-18)	10
DE	Gebrauchsanweisung (Revisionsstand: 2023-09-18)	12
EL	Οδηγίες χρήσης (Ημερομηνία αναθεώρησης: 2023-09-18)	14
ES	Instrucciones de uso (Última revisión: 2023-09-18)	16
ET	Kasutusjuhend (Viimati muudetud: 2023-09-18)	18
FI	Käyttöohje (Revisiostatus: 2023-09-18)	20
FR	Notice d'utilisation (Date de révision: 2023-09-18)	22
HR	Upute za uporabu (Stanje revizije: 2023-09-18)	24
HU	Használati utasítás (Aktualizálás időpontja: 2023-09-18)	26
IT	Istruzioni per l'uso (Ultima revisione: 2023-09-18)	28
KK	Пайдалану нұсқаулары (Редакция күйі: 2023-09-18)	30
KO	사용 설명서 (2023년 09월 수정)	32
LT	Naudojimo instrukcija (Tikrinta: 2023-09-18)	34
LV	Lietošanas instrukcija (Pārbaudes statuss: 2023-09-18)	36
NL	Gebruiksaanwijzing (Revisiestatus: 2023-09-18)	38
NO	Brugsanvisning (Revisjonsstatus: 2023-09-18)	40
PL	Instrukcja użytkowania (Ostatnia aktualizacja: 2023-09-18)	42

PT	Instruções de utilização (Data da última atualização: 2023-09-18)	44
RO	Instrucțiuni de utilizare (Stadiu revizie: 2023-09-18)	46
RU	Инструкция по применению (Ревизионный статус: 2023-09-18)	48
SK	Návod na použitie (Posledná revízia textu: 2023-09-18)	50
SL	Navodilo za uporabo (Datum revizije: 2023-09-18)	52
SV	Bruksanvisning (Revision status: 2023-09-18)	54
TH	คำแนะนำวิธีใช้ (แก้ไขปรับปรุงครั้งล่าสุดเมื่อ: 2023-09-18)	56
TR	Kullanma Talimatı (Bilgilerin son güncellenme tarihi: 2023-09-18)	58
ZH	使用说明 (修订状态: 2023 年 09 月)	第60页

PALAMIX® vacuum pump

1 General information

PALAMIX® vacuum pump.

1.1 Package content

Product	Description	Qty	Article number
PALAMIX® vacuum pump	Reusable vacuum pump with one way valve	1	66036748

2 Device description

The **PALAMIX® vacuum pump** is a compressed air operated vacuum pump. This device generates a vacuum in the mixing cartridge of the **PALAMIX® vacuum mixing system** or the **pro mixing systems**, which allows mixing of bone cement under vacuum.

The **PALAMIX® vacuum pump** is a non-sterile product, to be positioned on the floor of the operating theatre.

3 Directions for use

3.1 Intended purpose

PALAMIX® vacuum pump is intended to apply vacuum to **PALAMIX® uno/duo** or **pro mixing systems**.

3.2 Intended Patient Population

Patients requiring application of vacuum-mixed bone cement. Of note the **PALAMIX® vacuum pump** is neither in direct contact with the patient nor with the bone cement.

3.3 Intended User

Healthcare professionals in a clinical context and experienced in the handling of the product.

4 Indications

Applying vacuum during bone cement application with **PALAMIX® uno/duo** or **pro mixing systems** within its indica-

tions, such as cemented arthroplasty or reconstruction of bone, as well as for filling of suitable spacer molds.

5 Contraindications

There are no known contraindications.

6 Side effects and residual risks

6.1 Warnings

The **PALAMIX® vacuum pump** is located in the non-sterile area on the floor of the operating room.

6.2 Precautions

The **PALAMIX® vacuum pump** should not be used without a filter element. In the event of malfunctioning of the **PALAMIX® vacuum pump** contact your local Heraeus Medical representative.

6.3 Side effects

There are no known side effects.

6.4 Residual risks

There are no known residual risks.

7 Handling

The **PALAMIX® vacuum pump** is connected with the in-house compressed air supply. The functional range of the pump is an operating pressure between 5 and 10 bar (0.5–1.0 MPa) air pressure. In order to allow the use of the **PALAMIX® vacuum pump**, it has to be connected to the mixing cartridge of the **PALAMIX® vacuum mixing system** or **pro mixing systems**. For this purpose, the respective vacuum hose supplied has to be connected via the activated charcoal filter to the **PALAMIX® vacuum pump**. The **PALAMIX® vacuum pump** can only be used in conjunction with the **PALAMIX® vacuum mixing system** or the **pro mixing systems**. The respective operating steps are also in the corresponding instructions for use.

8 Maintenance and cleaning

Regular maintenance of the **PALAMIX® vacuum pump** is not required. The pump housing can be cleaned with a soft wipe without aggressive detergents. Take care here that no cleaning fluid or water enters the **PALAMIX® vacuum pump**.

9 Notes

In the event of malfunctioning of the **PALAMIX® vacuum pump** contact your local Heraeus Medical representative.

10 Disposal

Packaging material must be disposed of as clinical waste in compliance with local regulations. Loaned **PALAMIX® vacuum pumps** should be decontaminated and cleaned before being returned to Heraeus Medical GmbH. Instruments returned to Heraeus Medical GmbH should be accompanied by a certificate of decontamination.

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PALAMIX® is a trademark of Heraeus Medical GmbH. Made in Germany.