
Heraeus

Medical

PALAMIX®

Cement gun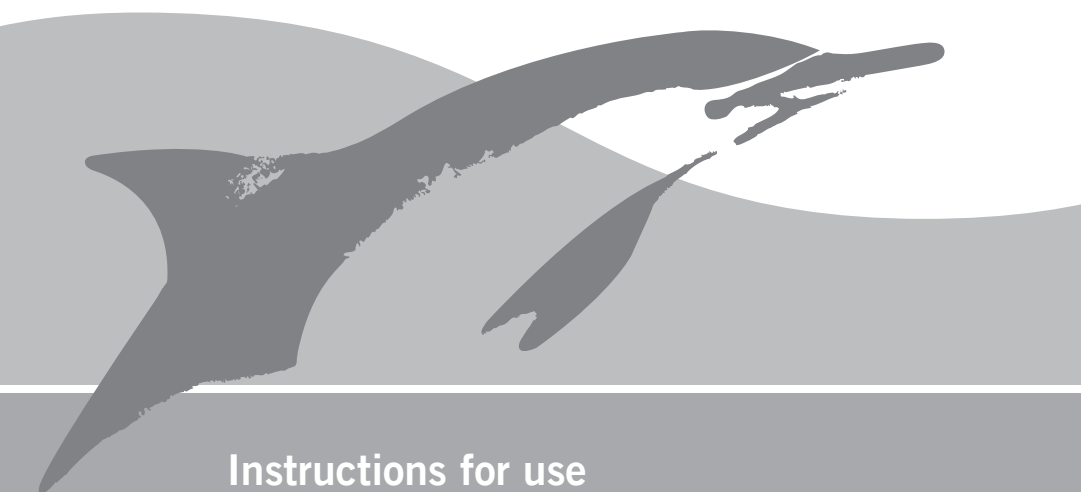

Instructions for use

PALAMIX® cement gun

EN	Instructions for use (Revision status: 2024-01-31)	4
BG	Инструкции за употреба (Дата на актуализиране на текста: 2024-01-31)	6
CS	Návod k použití (Poslední revise: 2024-01-31)	9
DA	Brugsanvisning (Aktualiseret: 2024-01-31)	11
DE	Gebrauchsanweisung (Revisionsstand: 2024-01-31)	13
EL	Οδηγίες χρήσης (Ημερομηνία αναθεώρησης: 2024-01-31)	16
ES	Instrucciones de uso (Última revisión: 2024-01-31)	19
ET	Kasutusjuhend (Viimati muudetud: 2024-01-31)	22
FI	Käyttöohje (Revisiostatus: 2024-01-31)	25
FR	Notice d'utilisation (Date de révision: 2024-01-31)	27
HR	Upute za uporabu (Stanje revizije: 2024-01-31)	30
HU	Alkalmazási utasítás (Aktualizálás időpontja: 2024-01-31)	33
IT	Istruzioni per l'uso (Ultima revisione: 2024-01-31)	36
KK	Пайдалану нұсқаулары (Редакция күні: 2024-01-31)	39
KO	사용 설명서 (2023년09월 수정)	42
LT	Naudojimo instrukcija (Tikrinta: 2024-01-31)	44
LV	Lietošanas instrukcija (Pārbaudes statuss: 2024-01-31)	46
NL	Gebruiksaanwijzing (Revisiestatus: 2024-01-31)	48
NO	Bruksinformasjon (Revisjonsstatus: 2024-01-31)	51

PL	Instrukcja użytkowania (Ostatnia aktualizacja: 2024-01-31)	53
PT	Instruções de utilização (Data da última actualização: 2024-01-31)	56
RO	Instrucțiuni de utilizare (Stadiu revizie: 2024-01-31)	59
RU	Инструкция по применению (Ревизионный статус: 2024-01-31)	62
SK	Návod na použitie (Posledná revízia textu: 2024-01-31)	65
SL	Navodilo za uporabo (Datum revizije: 2024-01-31)	67
SV	Bruksanvisning (Revision status: 2024-01-31)	70
TH	คำแนะนำวิธีใช้ (แก้ไขปรับปรุงครั้งสุดท้าย: 2024-01-31)	72
TR	Kullanma Talimatı (Bilgilerin son güncellenme tarihi: 2024-01-31)	75
ZH	使用说明 (修订状态: 2024 年 01 月)	第78页

PALAMIX® cement gun

1 General information

PALAMIX® cement gun.

1.1 Package content

Product	Description	Qty	Article number
PALAMIX® cement gun	Reusable gun	1	66036163

2 Device description

The **PALAMIX® cement gun** is a reusable orthopaedic application gun and is used in combination with the mixing cartridges of the **PALAMIX® vacuum mixing system** or the **pro mixing systems** in order to ensure precise and effortless application of bone cement.

3 Supporting Equipment

The **PALAMIX® cement gun** is intended to apply bone cement. It is suitable for use with the **PALAMIX® vacuum mixing system** or the **pro mixing systems** from Heraeus Medical GmbH. An overview of the products can be found on the website: www.heraeus-medical.com.

4 Directions for use

4.1 Intended purpose

The **PALAMIX® cement gun** is intended to accommodate the mixing cartridges of the **PALAMIX® vacuum mixing system** or the **pro mixing systems** and then to ensure precise application of bone cement.

4.2 Intended user

Healthcare professionals in a clinical context and experienced in the handling of the product.

5 Indications

The **PALAMIX® cement gun** is indicated to accommodate the mixing cartridges of the **PALAMIX® vacuum mixing system** or the **pro mixing systems** and then to ensure precise application of bone cement.

6 Contraindications

No contraindications are known for the use of the **PALAMIX® cement gun**.

7 Side effects and residual risks

7.1 Adverse events

No adverse events are known for the use of the **PALAMIX® cement gun**.

7.2 Warnings

If the **PALAMIX® cement gun** has been sterilized using steam, it must cool down before re-use. The **PALAMIX® cement gun** should only be used by trained personnel. The risk of hand injury or damage to protective gloves is possible due to improper handling. There is also a risk of damage to respective mixing cartridge if personnel are not trained correctly.


7.3 Precautions

Check the functionality of the **PALAMIX® cement gun** prior to each operation. If damaged, contact your local Heraeus Medical GmbH representative. The user must ensure the correct use of the **PALAMIX® cement gun**.

8 Handling

8.1 Operating instructions

The trigger of the **PALAMIX® cement gun** is operated by hand to expel the bone cement from the mixing cartridge. Activating the trigger advances the feed rod in the mixing cartridge and expels the bone cement under high pressure. If the cartridge jams on the gun after application of the bone cement, this is because the system is under tension. In this case, activate the trigger completely with the feed rod release mechanism at the back closed. Then loosen the feed rod release mechanism while the trigger is still under tension. The cartridge can then be removed.

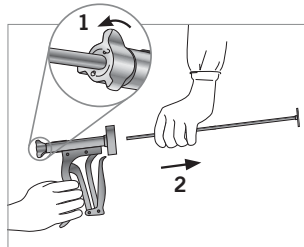To release, unlock the feed rod release mechanism (↺) and remove the feed rod.

8.2 Maintenance

Periodic maintenance of the **PALAMIX® cement gun** is not required.

9 Storage and sterilization

9.1 Storage

The **PALAMIX® cement gun** must be stored under dry and clean conditions. Care shall be exercised during transport and handling of the **PALAMIX® cement gun** to avoid any damage or alteration to the performance characteristics and it's packaging as received.

9.2 Cleaning and sterilization

The **PALAMIX® cement gun** is intended for multiple use and must be cleaned and sterilized each time prior to use. The feed rod must be separated from the grip part of the **PALAMIX® cement gun** for ideal cleaning and sterilization. The **PALAMIX® cement gun** should be cleaned and sterilized as for any conventional surgical instrument.

Firstly, clean off gross soiling from the product using a soft non-metallic brush, particularly down the lumen of the handpiece where the feed rod is inserted. Loosen organic residues with ultrasonic treatment and then rinse under running water. To sterilize the **PALAMIX® cement gun** in the steam autoclave, sterilize according to the following processing guidelines. If the **PALAMIX® cement gun** has been sterilized using steam, it must cool down before re-use.

10 Disposal

Packaging material must be disposed of as clinical waste in compliance with local regulations. Loaned

PALAMIX® cement guns should be decontaminated, cleaned, disinfected and sterilized before being returned to Heraeus Medical GmbH. Instruments returned to Heraeus Medical GmbH should be accompanied by a certificate of decontamination.

Reprocessing guidelines according to DIN EN ISO 17664

Instrument	PALAMIX® cement gun	
Ref.	66036163	
Manufacturer	Heraeus Medical GmbH Philipp-Reis-Str. 8/13 61273 Wehrheim, Germany www.heraeus-medical.com	
Cleaning and Disinfection		
Instrument can be cleaned automatically	<input checked="" type="checkbox"/> yes	
Cleaning agent	<input checked="" type="checkbox"/> alkaline	<input checked="" type="checkbox"/> pH neutral
Neutralisation (optional)	<input checked="" type="checkbox"/> phosphoric acid	<input type="checkbox"/> citric acid
Thermal disinfection	<input checked="" type="checkbox"/> at least 1 minute, 90 °C	
Ultrasonic cleaning (optional)	<input checked="" type="checkbox"/> yes	
Manual cleaning procedure	<ol style="list-style-type: none">1. The feed rod must be separated from the grip part of the cement gun.2. Firstly, clean off gross soiling using a soft non-metallic brush (i. e. tooth-brush) under running tap water (30 ± 5 °C) for at least 90 seconds.3. Loosen organic residues with 15 minutes of ultrasonic treatment.4. By using a medium-hard toothbrush remove apparent soil especially at windings, sliding surfaces and other hardly accessible parts under running tap water (30 ± 5 °C) for at least 60 seconds. At the same time move all mechanical elements to detach organic soil and check visually that the parts of the cement gun are absolutely clean.5. While rinsing pay particular attention to flushing the inner areas.6. Wipe down feed rod and the grip part using a lint free cloth.	
Sterilization		
The instrument must be sterilized?	<input checked="" type="checkbox"/> yes	
Minimum 6 minutes, gravity – no pre-vacuum (SAL 10 ⁻⁶)	steam	<input checked="" type="checkbox"/> 134 °C
Minimum 3 minutes, three pre-vacuum cycles (SAL 10 ⁻⁶)	steam	<input checked="" type="checkbox"/> 134 °C

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PALAMIX® is a trademark of Heraeus Medical GmbH. Made in Germany.