
Heraeus

# PALAMIX®

Cement gun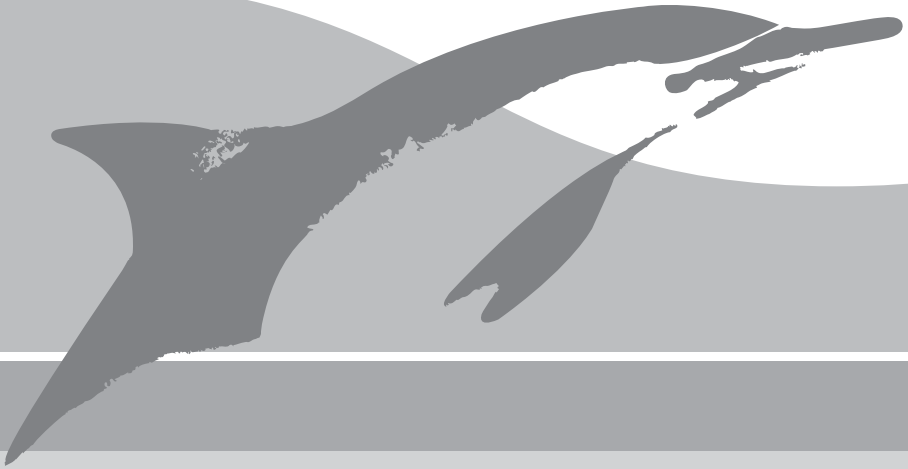

# PALAMIX® cement gun

EN	<b>Instructions for use</b> (Revision status: 2018-01)	4
EN (US)	<b>Note: Instructions for use for sale within the US. Instructions for use</b> (Revision status: 2018-04)	6
BG	<b>Инструкции за употреба</b> (Дата на актуализиране на текста: 2018-01)	8
CS	<b>Návod k použití</b> (Poslední revise: 2018-01)	10
DA	<b>Brugsanvisning</b> (Aktualiseret: 2018-01)	12
DE	<b>Gebrauchsanweisung</b> (Revisionsstand: 2018-01)	14
EL	<b>Οδηγίες χρήσης</b> (Ημερομηνία αναθεώρησης: 2018-01)	16
ES	<b>Instrucciones de uso</b> (Última revisión: 2018-01)	18
ET	<b>Kasutusjuhend</b> (Viimati muudetud: 2018-01)	20
FI	<b>Käyttöohje</b> (Revisiostatus: 2018-01)	22
FR	<b>Notice d'utilisation</b> (Date de révision: 2018-01)	24
HR	<b>Upute za uporabu</b> (Stanje revizije: 2018-01)	26
HU	<b>Alkalmazási utasítás</b> (Aktualizálás időpontja: 2018-01)	28
IT	<b>Istruzioni per l'uso</b> (Ultima revisione: 2018-01)	30
KK	<b>Пайдалану нұсқаулары</b> (Редакция күйі: 2018-04)	32
KO	<b>사용 설명서</b> (2018년 04월 수정)	34
LT	<b>Naudojimo instrukcija</b> (Tikrinta: 2018-01)	36
LV	<b>Lietošanas instrukcija</b> (Pārbaudes statuss: 2018-01)	38

NL	<b>Gebruiksaanwijzing</b> (Revisiestatus: 2018-01)	40
NO	<b>Bruksinformasjon</b> (Revisjonsstatus: 2018-01)	42
PL	<b>Instrukcja użytkowania</b> (Ostatnia aktualizacja: 2018-01)	44
PT	<b>Instruções de utilização</b> (Data da última actualização: 2018-01)	46
RO	<b>Instrucțiuni de utilizare</b> (Stadiu revizie: 2018-01)	48
RU	<b>Инструкция по применению</b> (Ревизионный статус: 2018-01)	50
SK	<b>Návod na použitie</b> (Posledná revízia textu: 2018-01)	52
SL	<b>Navodilo za uporabo</b> (Datum revizije: 2018-01)	54
SV	<b>Bruksanvisning</b> (Revision status: 2018-01)	56
TH	<b>คำแนะนำการใช้</b> (แก้ไขฉบับปรับปรุงครั้งสุดท้ายเมื่อ: 2018-01)	58
TR	<b>Kullanma Talimatı</b> (Bilgilerin son güncellenme tarihi: 2018-01)	60
ZH	<b>使用说明</b> (修订状态: 2018 年 10 月)	第62页

# PALAMIX® cement gun

## Description


The **PALAMIX® cement gun** is intended to accommodate the mixing cartridges of the **PALAMIX® vacuum mixing system** or the **PALACOS® R+G pro** and then to ensure precise and effortless application of bone cement.

The trigger of the **PALAMIX® cement gun** is operated by hand to expel the bone cement from the mixing cartridge. Activating the trigger advances the feed rod in the mixing cartridge and expels the bone cement under high pressure. The **PALAMIX® cement gun** can only be used in conjunction with the **PALAMIX® vacuum mixing system** or **PALACOS® R+G pro**. The respective operating steps are also described in detail as follows.

## Cleaning and sterilisation

The **PALAMIX® cement gun** is intended for multiple use and must be cleaned and sterilised each time prior to use.

The feed rod can be separated from the grip part of the **PALAMIX® cement gun** for ideal cleaning and sterilisation.

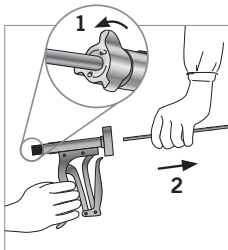To release, unlock the catch on the feed rod (1) and remove the rod. (2)

The **PALAMIX® cement gun** should be cleaned and sterilised as for any conventional surgical instrument. Firstly clean off gross soiling from the product using a soft non-metallic brush. Loosen organic residues with ultrasonic treatment and then rinse under running water. To sterilise the **PALAMIX® cement gun** in the steam autoclave, sterilise according to the following processing guidelines.

If the **PALAMIX® cement gun** has been sterilised with heat or steam, it must cool down before reuse.

## Maintenance

Periodic maintenance of the cement gun is not required.

## Notes

The **PALAMIX® cement gun** should only be used by trained personnel. The risk of hand injury or damage to protective gloves is possible due to improper handling. There is also a risk of damage to respective mixing cartridge if personnel are not trained correctly.

Check the functionality of the **PALAMIX® cement gun** prior to each operation. If damaged, contact your local Heraeus Medical representative. The user must ensure the correct use of the **PALAMIX® cement gun**.

**Processing guidelines according to EN ISO 17664**

Instrument	PALAMIX® cement gun	
Ref.	66036163	
Manufacturer	Heraeus Medical GmbH Philipp-Reis-Str. 8/13 61273 Wehrheim, Germany www.heraeus-medical.com	
<b>Cleaning and Disinfection</b>		
Instrument can be cleaned automatically	<input checked="" type="checkbox"/> yes	
Cleaning agent	<input checked="" type="checkbox"/> alkaline	<input checked="" type="checkbox"/> pH neutral
Neutralisation (optional)	<input checked="" type="checkbox"/> phosphoric acid	<input type="checkbox"/> citric acid
Thermal disinfection	<input checked="" type="checkbox"/> at least 1 min., 90°C	
Ultrasonic cleaning (optional)	<input checked="" type="checkbox"/> yes	
Manual cleaning procedure	<ol style="list-style-type: none"> <li>1. The feed rod must be separated from the grip part of the cement gun.</li> <li>2. Firstly clean off gross soiling using a soft non-metallic brush (i. e. toothbrush) under running tap water (<math>30 \pm 5^\circ\text{C}</math>) for at least 90 seconds.</li> <li>3. Loosen organic residues with 15 minutes of ultrasonic treatment.</li> <li>4. By using a medium-hard toothbrush remove apparent soil especially at windings, sliding surfaces and other hardly accessible parts under running tap water (<math>30 \pm 5^\circ\text{C}</math>) for at least 60 seconds. At the same time move all mechanical elements to detach organic soil and check visually that the parts of the cement gun are absolutely clean.</li> <li>5. While rinsing pay particular attention to flushing the inner areas.</li> <li>6. Wipe down feed rod and the grip part using a lint free cloth.</li> </ol>	
<b>Sterilisation</b>		
The instrument must be sterilised?	<input checked="" type="checkbox"/> yes	
Minimum 6 minutes, gravity – no pre-vacuum (SAL $10^{-6}$ )	steam	<input checked="" type="checkbox"/> 134°C
Minimum 3 minutes, three pre-vacuum cycles (SAL $10^{-6}$ )	steam	<input checked="" type="checkbox"/> 134°C

 Components made of stainless steel, handle as instruments, medical device class I. Bearings to be oiled.

EN (US)

# PALAMIX® cement gun

Note: Instructions for use for sale within the US.

## Description


The **PALAMIX® cement gun** is intended to accommodate the mixing cartridges of the **PALAMIX® vacuum mixing system** or the **PALACOS® R+G pro** and then to ensure precise and effortless application of bone cement.

The trigger of the **PALAMIX® cement gun** is operated by hand to expel the bone cement from the mixing cartridge. Activating the trigger advances the feed rod in the mixing cartridge and expels the bone cement under high pressure. The **PALAMIX® cement gun** can only be used in conjunction with the **PALAMIX® vacuum mixing system** or **PALACOS® R+G pro**. The respective operating steps are also described in detail as follows.

## Cleaning and sterilisation

The **PALAMIX® cement gun** is intended for multiple use and must be cleaned and sterilised each time prior to use.

The feed rod can be separated from the grip part of the **PALAMIX® cement gun** for ideal cleaning and sterilisation.

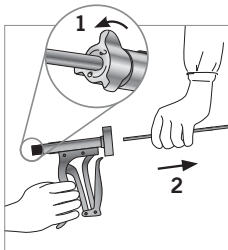To release, unlock the catch on the feed rod (↺) and remove the rod.

The **PALAMIX® cement gun** should be cleaned and sterilised as for any conventional surgical instrument. Firstly clean off gross soiling from the product using a soft non-metallic brush. Loosen organic residues with ultrasonic treatment and then rinse under running water. To sterilise the **PALAMIX® cement gun** in the steam autoclave, sterilise according to the following processing guidelines.

## Warnings

If the **PALAMIX® cement gun** has been sterilised with heat or steam, it must cool down before reuse. The **PALAMIX® cement gun** should only be used by trained personnel. The risk of hand injury or damage to protective gloves is possible due to improper handling. There is also a risk of damage to respective mixing cartridge if personnel are not trained correctly.

## Maintenance

Periodic maintenance of the cement gun is not required.

## Precautions

Check the functionality of the **PALAMIX® cement gun** prior to each operation. If damaged, contact your local Heraeus Medical representative. The user must ensure the correct use of the **PALAMIX® cement gun**.

## Caution

Federal law restricts this device to sale, distribution, and use by or on the order of a physician.

**Processing guidelines according to EN ISO 17664**

Instrument	PALAMIX® cement gun	
Ref.	66036163	
Manufacturer	Heraeus Medical GmbH Philipp-Reis-Str. 8/13 61273 Wehrheim, Germany www.heraeus-medical.com	
<b>Cleaning and Disinfection</b>		
Instrument can be cleaned automatically	<input checked="" type="checkbox"/> yes	
Cleaning agent	<input checked="" type="checkbox"/> alkaline	<input checked="" type="checkbox"/> pH neutral
Neutralisation (optional)	<input checked="" type="checkbox"/> phosphoric acid	<input type="checkbox"/> citric acid
Thermal disinfection	<input checked="" type="checkbox"/> at least 1 min., 90°C	
Ultrasonic cleaning (optional)	<input checked="" type="checkbox"/> yes	
Manual cleaning procedure	<ol style="list-style-type: none"> <li>1. The feed rod must be separated from the grip part of the cement gun.</li> <li>2. Firstly clean off gross soiling using a soft non-metallic brush (i. e. toothbrush) under running tap water (<math>30 \pm 5^\circ\text{C}</math>) for at least 90 seconds.</li> <li>3. Loosen organic residues with 15 minutes of ultrasonic treatment.</li> <li>4. By using a medium-hard toothbrush remove apparent soil especially at windings, sliding surfaces and other hardly accessible parts under running tap water (<math>30 \pm 5^\circ\text{C}</math>) for at least 60 seconds. At the same time move all mechanical elements to detach organic soil and check visually that the parts of the cement gun are absolutely clean.</li> <li>5. While rinsing pay particular attention to flushing the inner areas.</li> <li>6. Wipe down feed rod and the grip part using a lint free cloth.</li> </ol>	
<b>Sterilisation</b>		
The instrument must be sterilised?	<input checked="" type="checkbox"/> yes	
Minimum 6 minutes, gravity – no pre-vacuum (SAL $10^{-6}$ )	steam	<input checked="" type="checkbox"/> 134°C
Minimum 3 minutes, three pre-vacuum cycles (SAL $10^{-6}$ )	steam	<input checked="" type="checkbox"/> 134°C

 Components made of stainless steel, handle as instruments, medical device class I. Bearings to be oiled.

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PALAMIX® is a trademark of Heraeus Medical GmbH. Made in Germany.